

What's New at Central!



**SIMPLE, QUICK
AND EASY TO
USE TEST KIT**

**NO MORE PAINFUL SAMPLING - EASY AND RELIABLE
ANTIGEN TEST FROM SALIVA (MOUTH FLUIDS) SWAB.**





- ✓ 95.65% SENSITIVITY, 98.44% SPECIFICITY
- ✓ SAMPLE TAKEN FROM MOUTH FLUID (SALIVA)
- ✓ FAST RESULT AFTER 10 MINUTES

ABOUT THE SALIVA (LOLLIPOP) TEST

V-Chek™ 2019-nCoV Saliva Rapid Test (Lollipop Test)

Product Code: VCS-20

The Test Card is a lateral flow immunoassay intended for the qualitative detection of nucleocapsid protein antigen from 2019-nCoV in saliva specimens directly collected from individuals who are suspected of COVID-19 by their healthcare provider within the first 7 days of symptom onset.

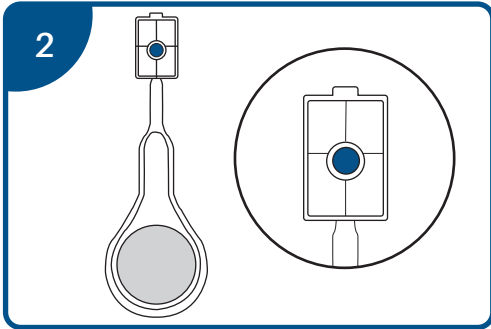
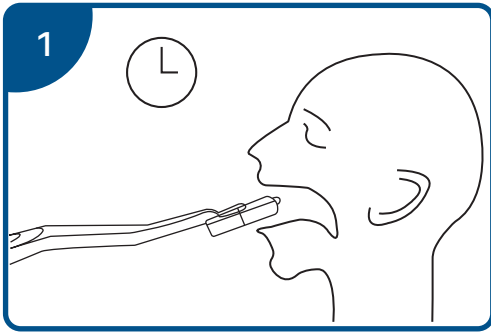


TEST PROCEDURE

Before test, please read the instructions carefully.

1. Take the test card to equilibrate to room temperature.
2. Unpack the aluminium foil bag, place the test card horizontally on the table and mark it.
3. Insert the saliva swab into the test card holder and push down saliva swab.
The bump at the end of the saliva swab must be into the hole of the test card holder.

4. As the test begins to work, the purple colour moves across the result window in the center of the test device.
5. Wait for 10 minutes and read the results.

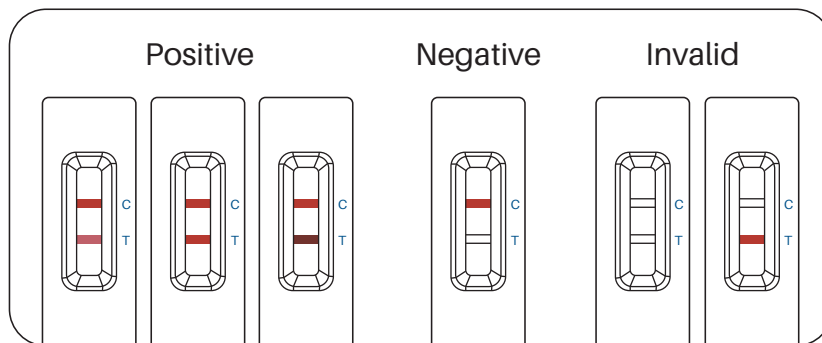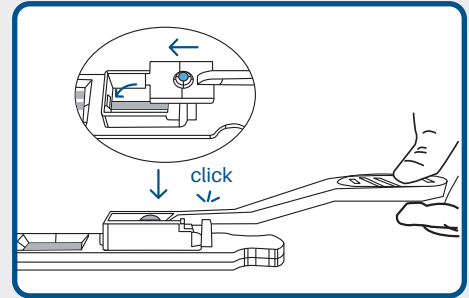

MATERIALS INCLUDED

- ✓ TEST CARD
- ✓ SALIVA SWAB
- ✓ INSTRUCTIONS FOR USE
- ✓ QUICK REFERENCE INSTRUCTIONS



Contact all branches

p: **1300 347 347** | e: sales@centralcleaning.com.au
www.centralcleaning.com.au

Victoria | New South Wales | Tasmania | South Australia | Queensland | Western Australia